BRUKSANVISNING (Versjon 1.0)

IntegraMouse Plus

inklusive tilbehør

CE

Norsk

KJÆRE KUNDE!

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Guidance and manufacturer's declaration – electromagnetic emissions		
The IntegraMouse Plus and its accessories are intended for use in the electromagnetic environment specified below. The customer or the user of the IntegraMouse Plus and its accessories should assure that they are used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 22	Class B	The IntegraMouse Plus and its accessories are suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Guidance and manufacturer's declaration – electromagnetic immunity

The IntegraMouse Plus and its accessories are intended for use in the electromagnetic environment specified below. The customer or the user of the IntegraMouse Plus and its accessories should assure that they are used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV for power supply lines ±1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	±1 kV differential mode ±2 kV common mode	
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % UT (>95 % dip in UT) for 0,5 cycle 40 % UT (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles <5 % UT (>95 % dip in UT) for 5 s	<5 % UT (>95 % dip in UT) for 0,5 cycle 40 % UT (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles <5 % UT (>95 % dip in UT) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the IntegraMouse Plus and its accessories requires continued operation during powermains interruptions, it is recommended that the IntegraMouse Plus and its accessories be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	The power frequency magnetic field should be measured in the intended installation location to assure that it is sufficiently low.

NOTE U_T is the a.c. mains voltage prior to application of the test level.

Guidance and manufacturer's declaration – electromagnetic immunity

The IntegraMouse Plus and its accessories are intended for use in the electromagnetic environment specified below. The customer or the user of the IntegraMouse Plus and its accessories should assure that they are used in such an electromagnetic environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
<p>Conducted RF IEC 61000-4-6</p> <p>Radiated RF IEC 61000-4-3</p>	<p>3 V/m</p> <p>80 MHz to 2,5 GHz</p>	<p>3 V/m</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the IntegraMouse Plus and its accessories, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> <p>$d = 1,2\sqrt{P}$ 150 kHz to 80 MHz</p> <p>$d = 1,2\sqrt{P}$ 80 MHz to 800 MHz</p> <p>$d = 2,3\sqrt{P}$ 800 MHz to 2,5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> <div style="text-align: center;">  </div>

NOTE 1 At 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the IntegraMouse Plus and its accessories are used exceeds the applicable RF compliance level above, the IntegraMouse Plus and its accessories should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the IntegraMouse Plus and its accessories.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the IntegraMouse Plus and its accessories

The IntegraMouse Plus and its accessories are intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the IntegraMouse Plus and its accessories can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the IntegraMouse Plus and its accessories as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter		
	M		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 An additional factor of 10/3 is used in calculating the recommended separation distance for transmitters in the ISM frequency bands between 80 MHz and 2,5 GHz to decrease the likelihood that mobile/portable communications equipment could cause interference if it is inadvertently brought into patient areas.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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LifeTool Solutions GmbH
Hafenstraße 47-51, A-4020 Linz
AUSTRIA
E: office@lifetool-solutions.at
W: http://www.integramouse.com

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We declare under sole responsibility that the product including accessories described above as delivered are in compliance with directive 93/42/EEC and the Austrian medical-device-law BGBl. 657/1996 in the presently valid edition. The products are CE marked.

Linz, 09.12.2011

Mag. David Holzer
Geschäftsführer / CEO
LifeTool Solutions GmbH
A-4020 Linz, Hafenstr. 47-51
Tel +43 (0) 732 9015 5270, Fax 5271
office@lifetool-solutions.at
www.lifetool-solutions.at