

# SETTING UP THE QWAYO FOR THE FIRST TIME (PAGE 1)

To enable the Qwayo Tablet to connect to the Wi-Fi please have the Wi-Fi Passcode available before proceeding. (Usually located on a sticker on the Router)

## 1) Connect the tablet to its power supply


Insert the USB cable into the Charging Head




Insert the USB cable into the tablets Charger / Accessory Port




## 2) Turn the tablet on by pressing and holding the Power/ Lock Key for 3 seconds

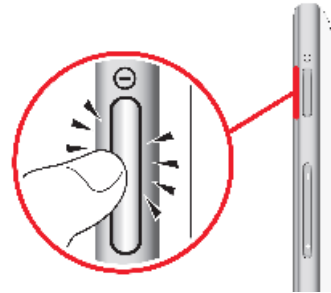

## 3) Once the splash screen has appeared swipe the screen to unlock then wait

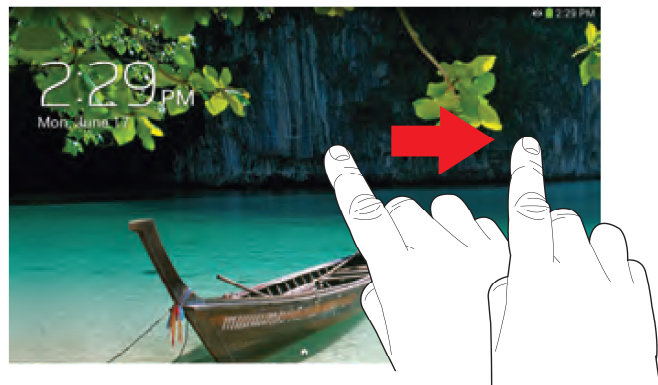


## 4) The Qwayo software will start-up automatically as shown below

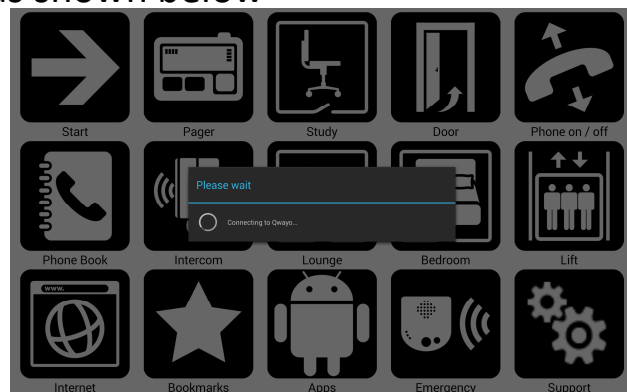

## 5) Press the back arrow shown



to get back to the home screen as shown below

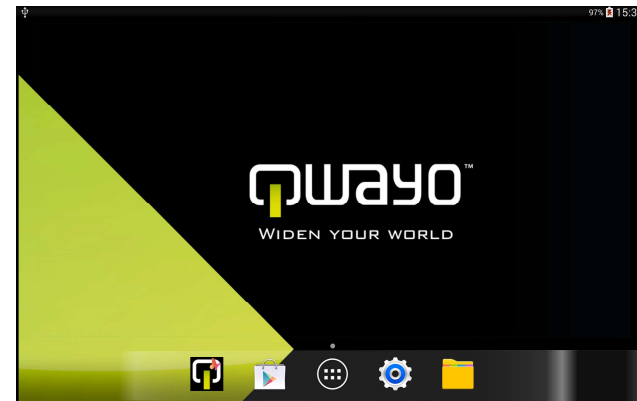


## 6) Press the Settings icon




## 7) Turn on the Wi-Fi setting (by tapping on/off switch)



## 8) After a short time a list of Wi-Fi Networks will be shown. Press the access point you wish to connect the tablet to

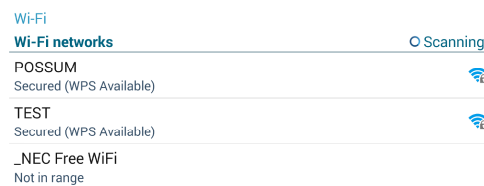

## 9) If the network has a security passcode please enter this and click connect

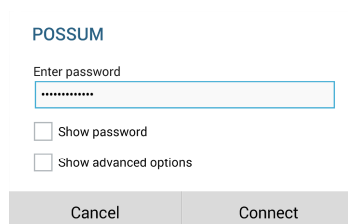

## 10) After a short time you should see the Wi-Fi icon appear in the top right of the screen



## 11) Once the Wi-Fi is confirmed as connected, turn on Bluetooth (By tapping on/off switch)



## 12) Remove battery cover on bottom of Qwayo Hub

To remove the underside cover, push the release latch in the direction shown with the green arrow in the picture on the left.


If the underside cover does not pop open there is a small ledge on the release latch that can be levered by using your fingernail to aid the cover opening.



## 13) Locate and turn On Power

Once the underside compartment has been removed you will have access to the On/Off switch.

Slide the switch towards the rotary dial to turn the Qwayo Hub on.

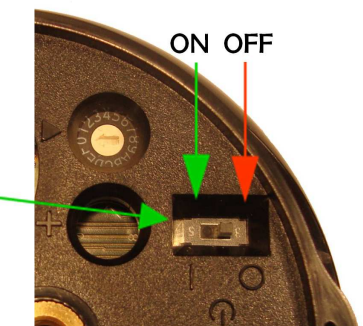

When turning the Qwayo Hub on, the Qwayo Logo (on the top of the Qwayo Hub) will light up green & then flash green 5 times.

## 14) Replace Battery cover

**PLEASE TURN TO THE NEXT PAGE**

# SETTING UP THE QWAYO FOR THE FIRST TIME (PAGE 2)

## 15) Plug Qwayo Hub into charger



Insert the USB cable into the Hub's  
Charger Port


Indicates the Qwayo Hub is charging




Indicates the Qwayo Hub is charged



## 16) Reset the tablet by Pressing and holding the Power / Lock Key

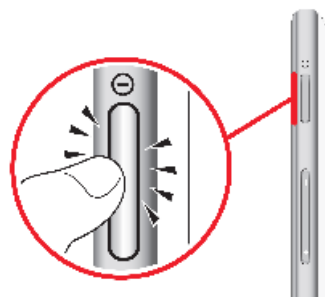


## 17) Once the tablet options have appeared press Restart

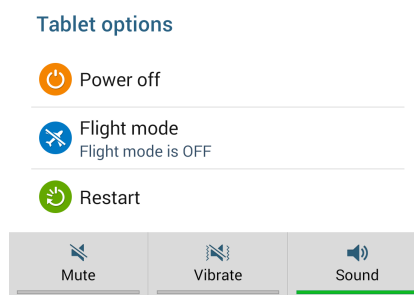


## 18) The system is now configured

## QWAYO TABLET

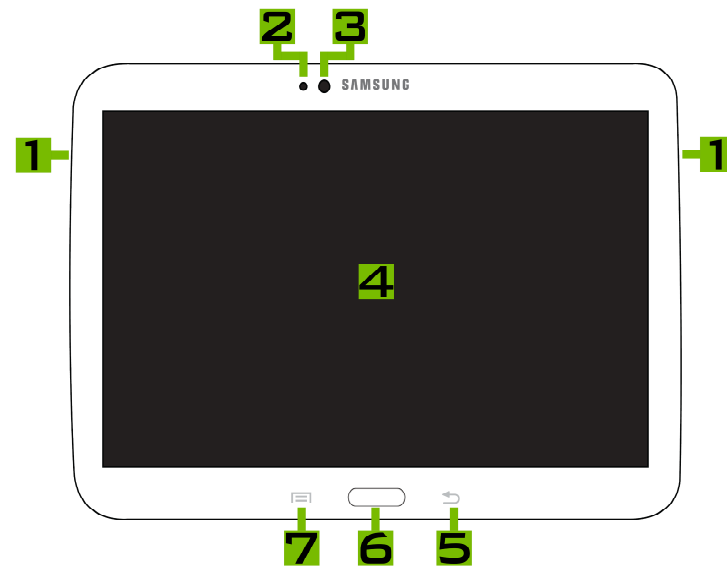

- 1. External Speakers
- 2. Light Sensor
- 3. Front-facing Camera Lens
- 4. Display Screen
- 5. Back
- 6. Home
- 7. Menu
- 8. Power/Lock Key
- 9. Volume Control
- 10. Memory Card Slot

## QWAYO HUB

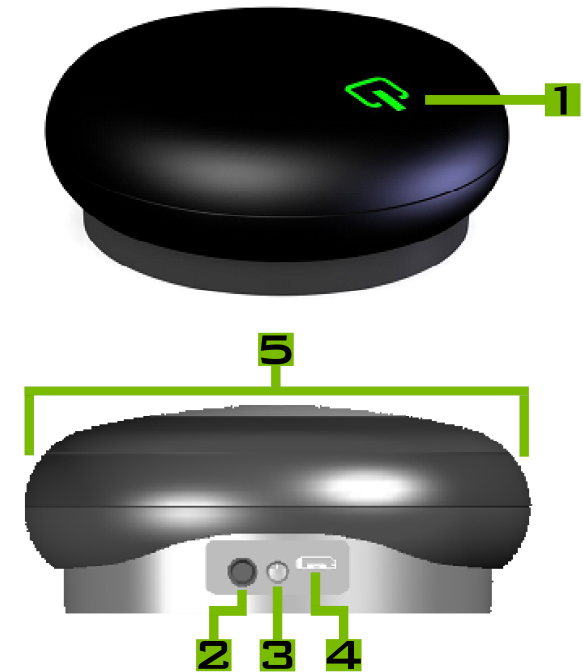

- 1. Indicator LED
- 2. 3.5mm Switch Input
- 3. Power/ Charge LED
- 4. Charging Input
- 5. Switch (whole surface)

\* pressing force required to operate internal hub  
switch = 55 grams / 0.54 N



FOR TECHNICAL SUPPORT

YOU CAN VISIT: [WWW.QWAYO.CO.UK](http://WWW.QWAYO.CO.UK)

EMAIL: [QWAYO@POSSUM.CO.UK](mailto:QWAYO@POSSUM.CO.UK)

OR CALL: 01296 461000

Possum Ltd  
8 Farmbrough Close  
Stocklake Park Industrial Estate  
Aylesbury  
Bucks  
HP20 1DQ  
Phone: 44+ (0) 1296 461000  
E-mail: [qwayo@possum.co.uk](mailto:qwayo@possum.co.uk)  
Web: [www.possum.co.uk](http://www.possum.co.uk)