


User manual for Bucinator PAULUS +

Paulus+

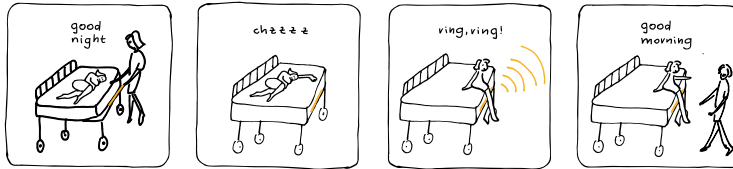

1. Safety instructions

NOTE

Before starting up and using Bucinator, these instructions must be read completely. Keep the manual and reuse it if needed. In the event of product, property or personal damage caused by non-observance of these instructions, the guarantee expires.

HINWEISE ZUR BENUTZUNG

- The Bucinator system may not be declared as a safety or medical product.
- The Bucinator system does support the nursing staff; it does **not** replace it.
- Before every use the Bucinator system must be checked for its functionality.

2. Product description

COMPOSITION

Bucinator Paulus + consists of a sensor strip and a protective cover.

Red LED to monitor function and battery life.



HANDLING

1. Place a Bucinator pad underneath the mattress on the side in the exit area.
2. Connect the receiver to the nurse call system.
3. Test the systems functionality by triggering a test alarm.

3. Cleaning/disinfection

The system can be cleaned with all commercially available disinfectants and cleaning agents, max. 70% alcohol by volume.

Do not use any caustic agents!

4. Battery management

Depending on how much the system is in use, the button cell will last for several years.

If the LED light continues to flash for approx. 3 seconds, please change the battery.

Proceed as follows:

4. Take the device out of the protective cover.
5. Loosen the screws on the housing with a screwdriver and open the housing.
6. If necessary, remove the old battery and insert the new one. Use only CR2032 batteries. Pay attention to the correct polarity.
7. Lock the battery compartment and check the systems functionality.

ATTENTION!

Keep the batteries out of the reach of children!

Empty batteries are hazardous waste and do not belong in the household waste!